

Ribbon > Home tab > Viewport group > Apply brightness and contrast

Button identification




Associated command

brightness-contrast-dropdown

Function description

This button brings down a dropdown menu with a number of default settings for brightness and contrast.

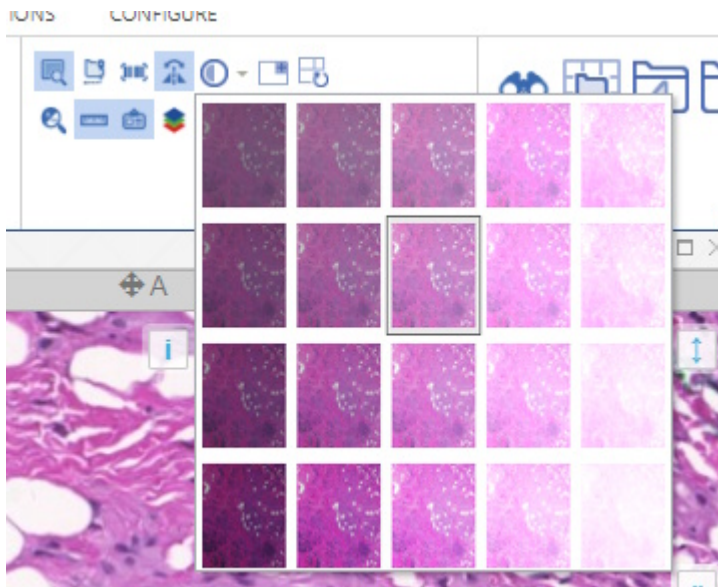

Click the thumbnail to apply the effect that you want. Thumbnails on the left show less brightness, and are brighter on the right. Thumbnails on the top show less contrast, and more contrast on the bottom.

Related functionality

[Ribbon > Home tab](#)

From: <https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link: https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=ribbon_home_viewport_brightness_contrast&rev=1676037896

Last update: **2023/02/10 17:04**

