

Ribbon > Home tab > Viewport group > Toggle slide label

Button identification




Associated command

viewport-toggle-label

Function description

This function shows or hides the full path reference to the current slide in the top-right area of the ribbon

When enabled, the ribbon looks like this:

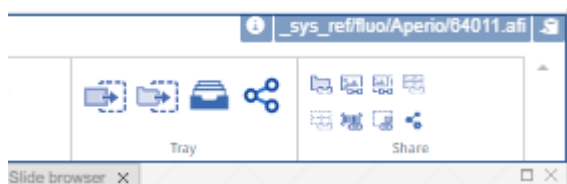

When disabled, the widget is hidden from the ribbon:



Unlike other toggle functionality, this one does not impact the viewport area

Related functionality

[Ribbon > Home tab](#)

From:
<https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link:
https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=ribbon_home_viewport_toggle_label

Last update: **2023/02/10 20:39**

