

# Working with grids

PMA.studio uses viewport as a grid system. UI provides a separate tab “GRID” to perform grid operations on the viewport.




“GRID” tab consists of 6 sections:

- 1. [Size](#)
- 2. [Synchronize](#)
- 3. [R/C](#) (rows and columns)
- 4. [Slide viewing properties](#)
- 5. [Snapshot](#)
- 6. [Share](#)

## Size

Size section allows you to break viewport as a grid system with different sizes. This includes grid sizes of 1x1,2x2,3x3 and even you can customized the grid size according to your need.

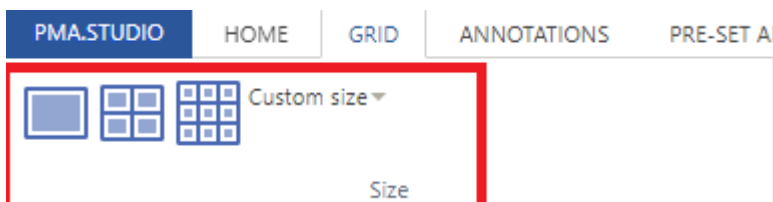

### (1x1 Grid size)

First grid option allows you to divide the viewport into 1x1 column grid.



### (2x2 Grid size)

Second grid option divides the viewport into 2x2 column grid.



### (3x3 Grid size)

Third grid option divides the viewport into 3x3 column grid.



### (Custom Grid size)

You can even customize the grid according to your need by using the custom grid option.



## Synchronize

This section allows you to perform different actions as a whole on individual grid column or as a whole grid system. It is further divided into 4 sub actions:



### Toggle synchronous navigation between cells in the current grid

This button allows you to perform set of synchronous actions on whole grid.

### Clear all grid cells

This button allows clearing all the grid cells at the same time

### Clear current cell only

This button allows to clear the selected grid cell only

### Open current slide into separate viewport panel

This button opens the current selected slide into a separate viewport tab



### RC


This section allows you to add or remove rows/columns to a grid.



## Slide


This section consists of set of tools that includes:

1. Changing contrast/exposure of selected slide
2. Rotation of slide
3. Pop out selected slide
4. Send slide to Tray
5. Zoom in and Zoom out selected slide

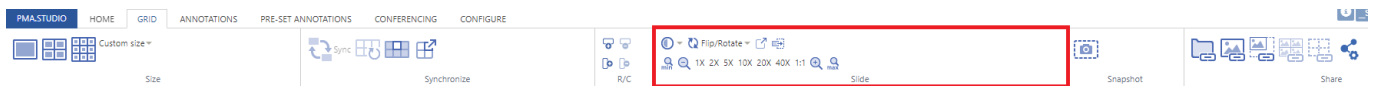


### (Changing contrast of selected slide)

This tool allows you to change contrast of selected slide.

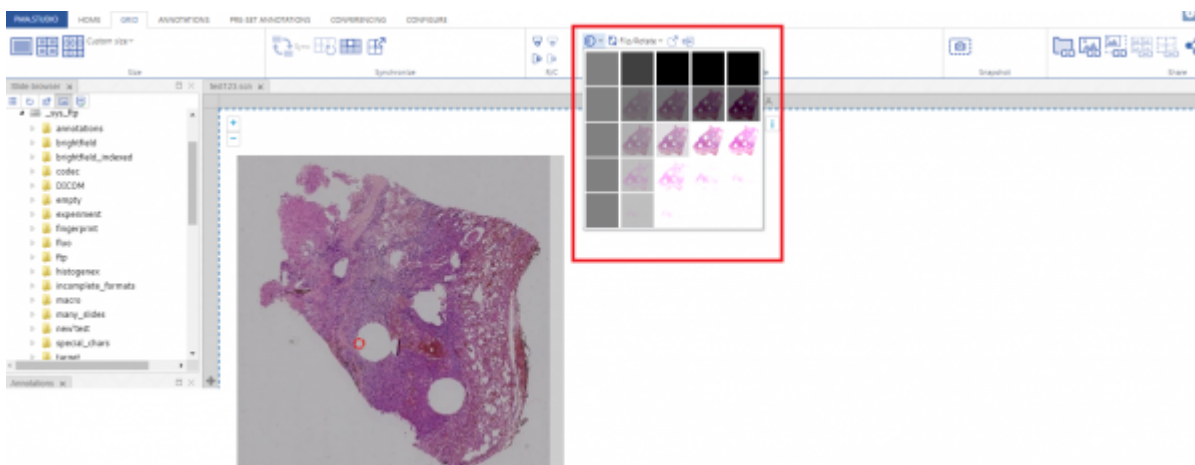


### (Rotation of slide)

This tool allows you to rotate slide as per your convenience.

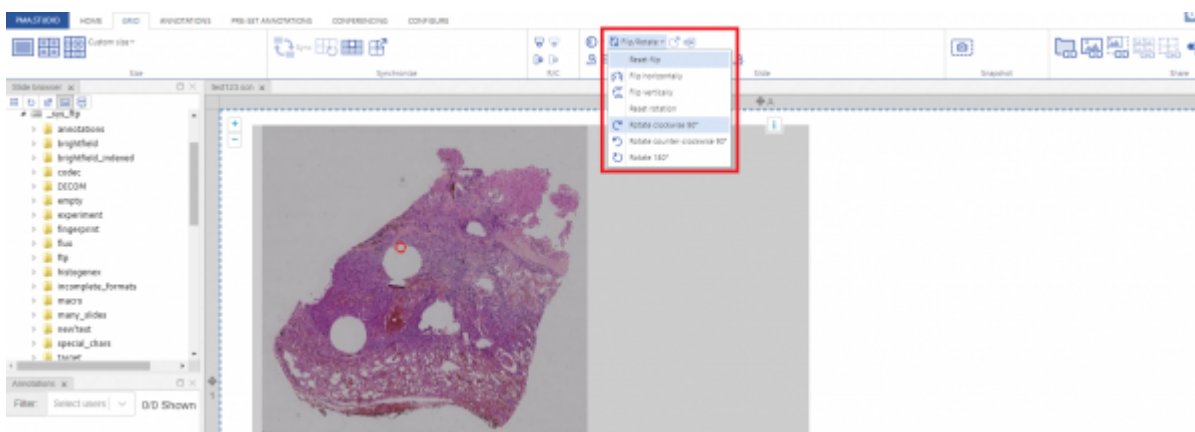


### (Pop out selected slide)

Once you click on this tool it generates a shared link for slide and opens it in a new window to view.

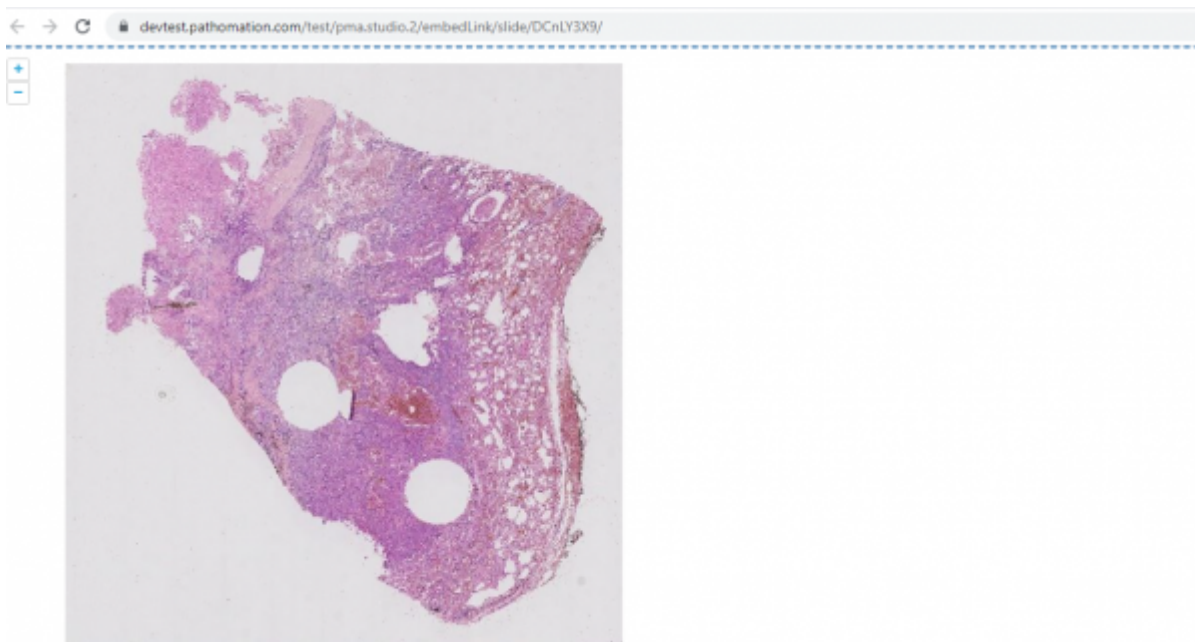


**(Send slide to Tray)**

This tool sends the selected slide to Tray panel.

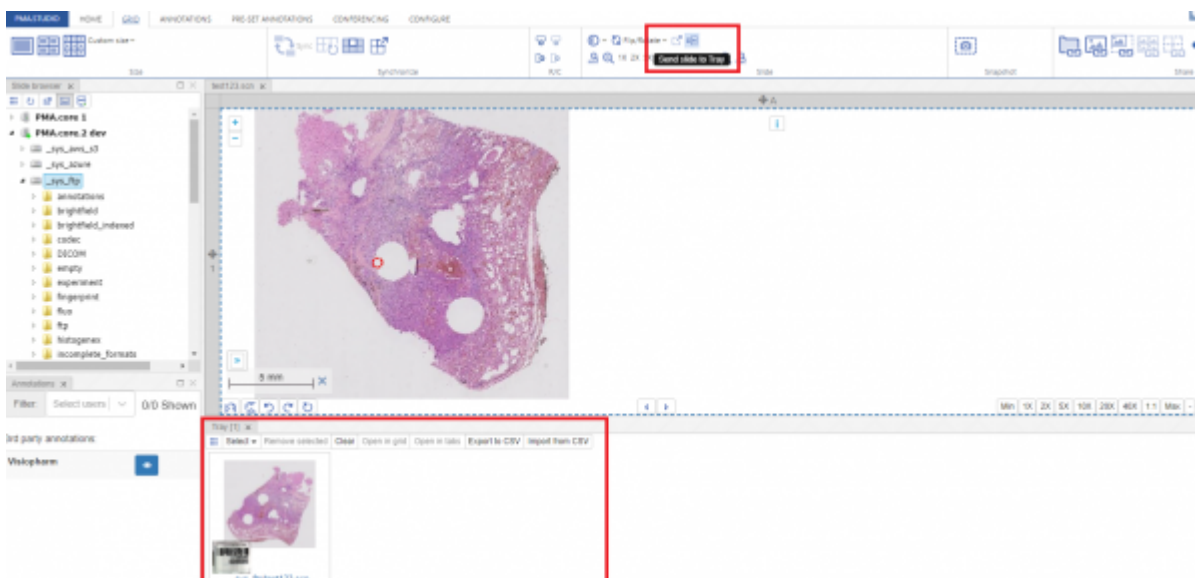

**(Zoom in and Zoom out)**

PMA.studio provides multiple zooming options to zoom-in and zoom-out selected slide.

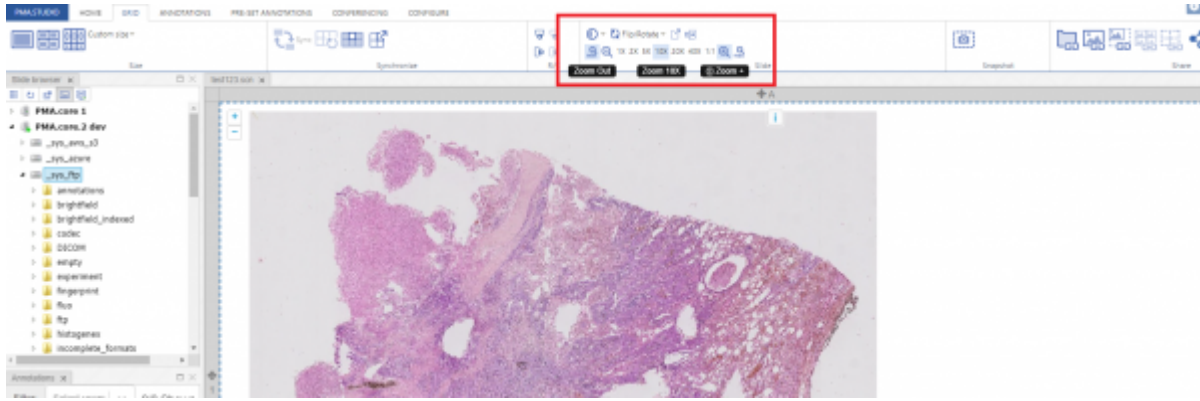

## Snapshot

## Share

From:  
<https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link:  
[https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=slides\\_grid&rev=1674046192](https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=slides_grid&rev=1674046192)

Last update: **2023/01/18 15:49**

