


Sharing live views

Sharing content is arguably one of the most important applications of digital pathology, if not for the Web in general. PMA.studio provides multiple options to share a slide or a folder or region of interest(ROI) as per your convenience. You can even create a custom share link with expiration date and password.

When you click either of share a slide or slide folder(multiple slides) or region of interest(ROI) buttons, it generates a shared link and shows a popup window that allows you to copy embedded viewer url that you can paste in browser and view the slide/folder or download the QR-code.

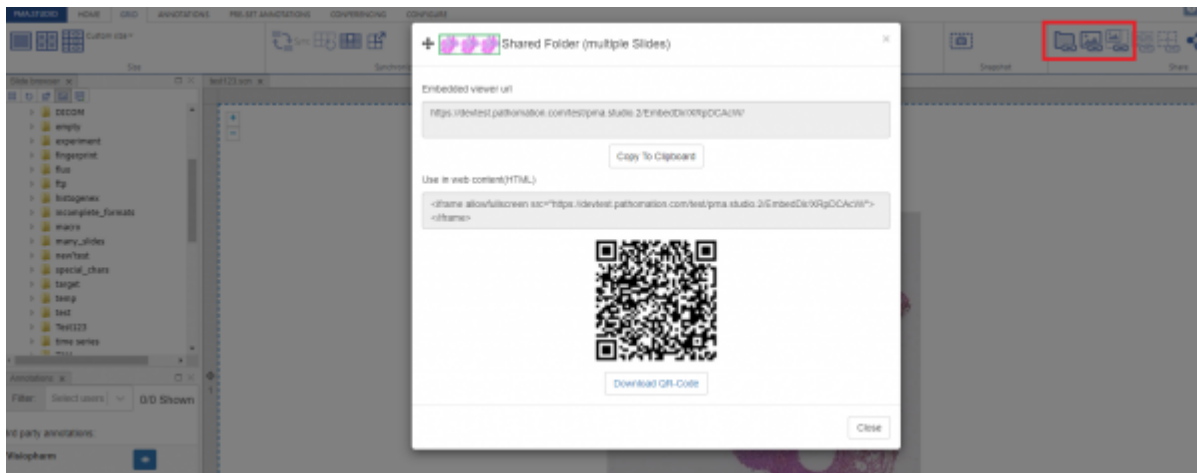

From:
<https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link:
https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=slides_sharing&rev=1651076828

Last update: **2022/04/27 19:27**

