



# Sharing live views

Sharing content is arguably one of the most important applications of digital pathology, if not for the Web in general. PMA.studio provides multiple options to share a slide or a folder or region of interest(ROI) as per your convenience. You can even create a custom share link with expiration date and password.


When you click either of share a slide or slide folder(multiple slides) or region of interest(ROI) buttons, it generates a shared link and shows a popup window that allows you to copy embedded viewer url that you can paste in browser and view the slide/folder or download the QR-code.

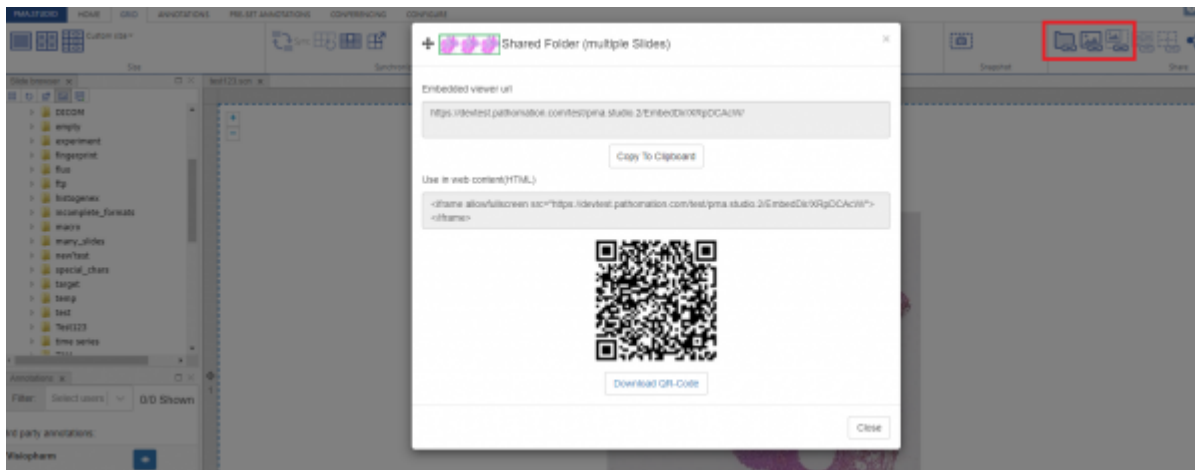


If you click on the custom share link button it will show a popup box with multiple options to selected before generating the slide/folder/ROI link. It also provides you option to include barcode, show annotation besides adding password access to the link generated. You can either create the link from this popup window or even preview the link generated.

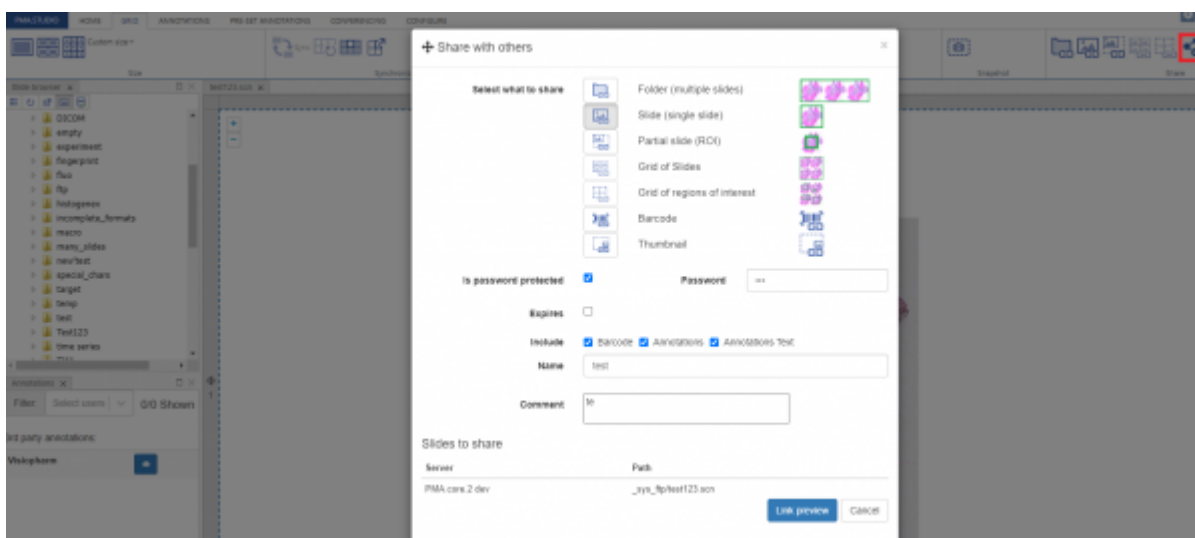



## Password protected preview

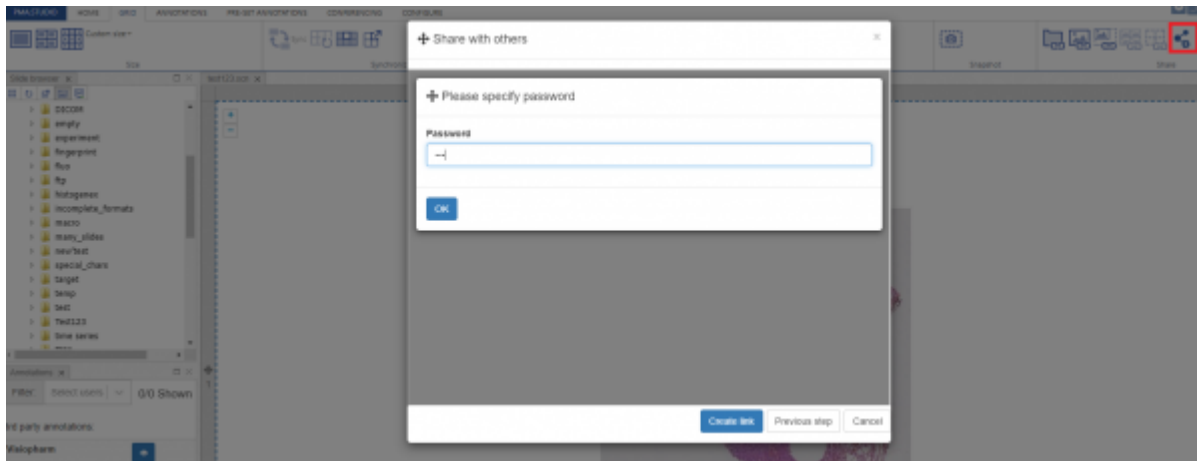

## Preview

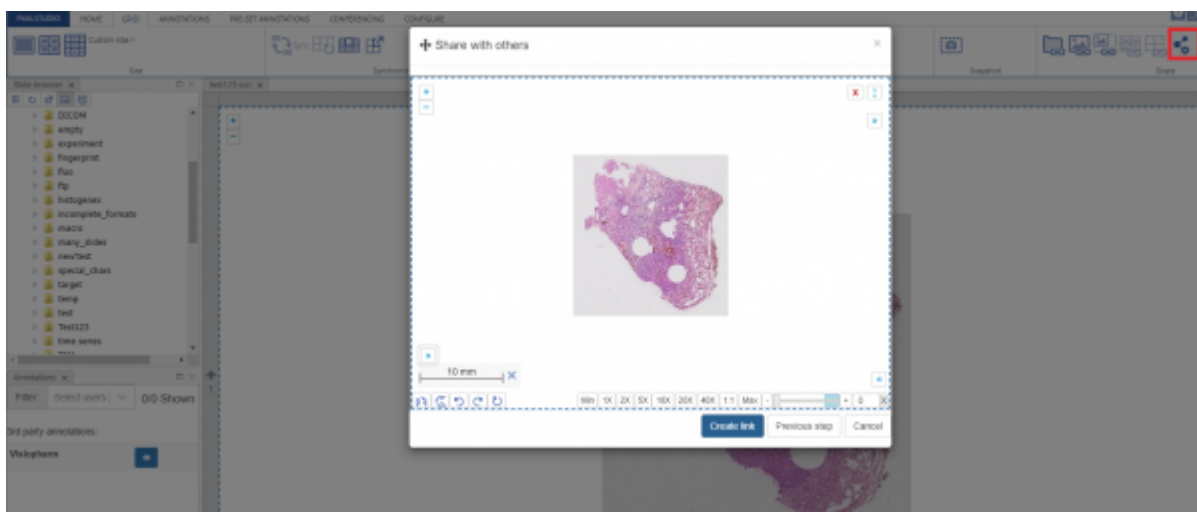

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And once you press the "Create Link" button it will generate the share link and show a popup window with option to "Copy to clipboard" or "Download QR-Code"

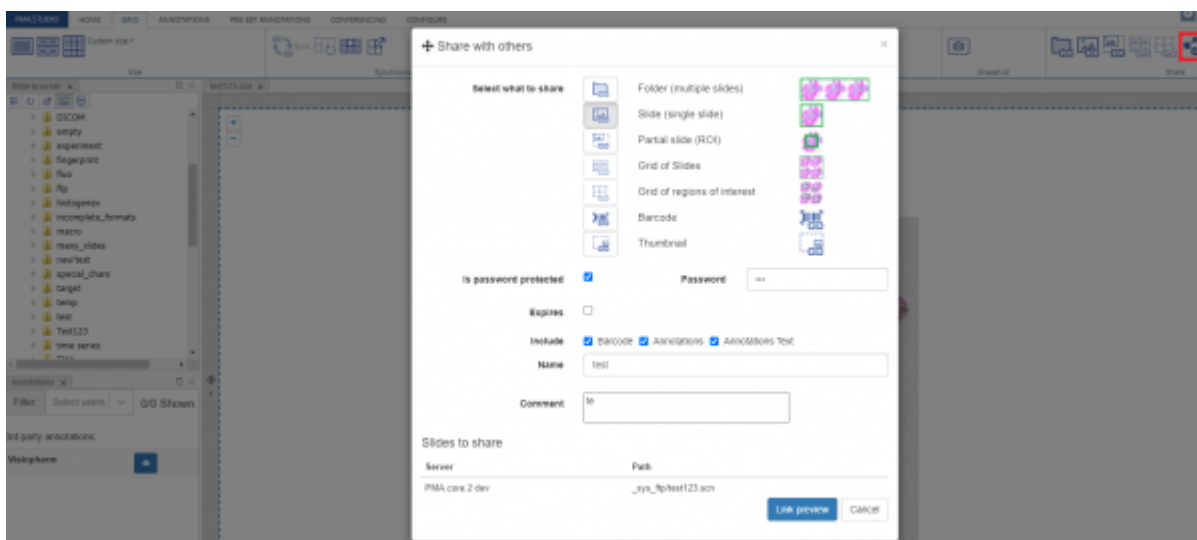

From:  
<https://docs.pathomation.com/pma.studio/2.1.0/> - **PMA.studio 2.1.0**

Permanent link:  
[https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=slides\\_sharing&rev=1651077512](https://docs.pathomation.com/pma.studio/2.1.0/doku.php?id=slides_sharing&rev=1651077512)

Last update: **2022/04/27 19:38**

