


For PMA.core :

Host: Username: Password: Folder: 

```

status: Trying to connect...
status: Resolving address of [redacted]
status: Connecting to [redacted]
status: PMA.core v2.0.0.1686
status: Connection established...
status: Loading folder on PMA.core...
status: Folder successfully loaded...

```

- Host : PMA.core instance to connect to, it should be in this form "http(s):xxxxx/pma.core.2/" * Username & Password : connection credentials * Folder : Remote directory on server to navigate to if connection is successful

For My Pathomation :

Email: Password: Folder: 

```

status: Trying to connect to the cloud...
status: Resolving address of [redacted]
status: Connecting to [redacted]
status: PMA.core v2.0.0.1696
status: Connection established...
status: Loading user's folder on the cloud...
status: Folder successfully loaded...

```

* Email : The email address used to subscribe to Pathomation cloud. * Password : Password received on inbox once the subscription & payment are validated. * Folder : disabled text field, end user can only connect to its own subscription folder.

From: <https://docs.pathomation.com/pma.transfer/> - **PMA.transfer**

Permanent link: https://docs.pathomation.com/pma.transfer/doku.php?id=connection_bar_logging_text_area&rev=1589528445

Last update: **2020/05/16 05:14**

