

	Slidename	Slidesize	FileType	Last Modified
↑	..			
📁	Intel		File folder	
📁	local samples		File folder	
			Upload folder to remote site	
📁	PerfLogs			
📁	Program Files		File folder	
📁	Program Files (x86)		File folder	
📁	scripts		File folder	
📁	Shared folder		File folder	
📁	Slide Samples		File folder	
📁	SvmCache		File folder	

Initiated via mouse right click within the table view borders, it's composed of one menu item :  
"Upload folder to remote site".

Upload folder to remote site :

Has been be discussed in details earlier on this tutorial.

From:  
<https://docs.pathomation.com/pma.transfer/> - **PMA.transfer**

Permanent link:  
[https://docs.pathomation.com/pma.transfer/doku.php?id=context\\_menu\\_for\\_folders&rev=1589529398](https://docs.pathomation.com/pma.transfer/doku.php?id=context_menu_for_folders&rev=1589529398)

Last update: **2020/05/16 05:14**

