


| Slidename         | Slidesize  | FileType  | Last Modified                      |
|-------------------|------------|-----------|------------------------------------|
| ..                |            |           |                                    |
| 3D_FL_test01.mrxs | 43,620,186 | MRXS File | Thu Dec 26 06:34:59 GMT+01:00 2019 |
| clear.png         | 274        | PNG File  | Sun Dec 08 19:02:43 GMT+01:00 2019 |
| JP2K-33003-1.svs  |            |           | 05:40:22 GMT+01:00 2018            |
| JP2K-YCBCR-1.svs  |            |           | 06:36:50 GMT+01:00 2019            |
| sampleX.jpg       |            |           | 14:37:58 GMT+01:00 2019            |
| Test2.jpg         |            |           | 19:02:37 GMT+01:00 2019            |

Open slide in PMA.start  
Upload slide to remote site  
Inspect slide           Ctrl+Alt+I  
Copy full path           Ctrl+Alt+C

Initiated via mouse right click within the table view borders, it's composed of four menu items : "Open slide in PMA.start", "Upload slide to remote site", "Inspect slide" and "Copy full path".

Open slide in PMA.start :

Opens the slide in PMA.start

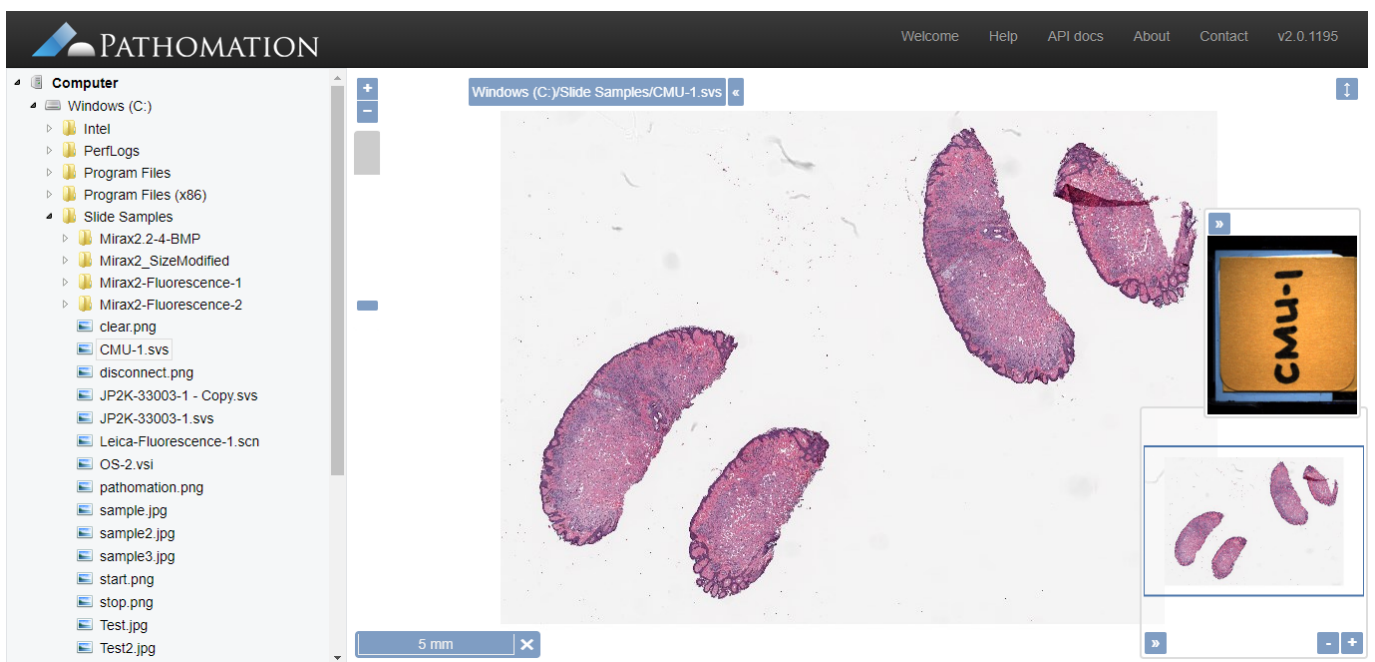

Upload slide to remote site :

Has been discussed in details earlier on this tutorial.

Inspect slide :

Information data regarding the slide (Thumbnail, fingerprint, size...)

|                        |                      |
|------------------------|----------------------|
| File Name              | C:\Slide Samples\Mir |
| Width                  | 10,752 pixels        |
| Height                 | 13,568 pixels        |
| Max Zoom Level         | 5                    |
| Micrometes per pixel X | 0.3225               |
| Micrometes per pixel Y | 0.3225               |
| Physical size          | 65,782,837 bytes     |
| Tile size              | 512                  |
| Finger print           | 5znbrPH9MGiY_YM84    |
| Barcode text           | [25058c19]           |
| Layers                 | 1                    |
| Channels               | 3                    |
| File format            | 3DHistech MRXS       |

The preview area shows a slide with a barcode and the following text:  
25058c19  
p231 500µm  
RQ5068 AcH3  
c4 J&J 25058

**Copy full path :**  
Copies the full path of the local slide to the clipboard.

From:  
<https://docs.pathomation.com/pma.transfer/> - **PMA.transfer**

Permanent link:  
[https://docs.pathomation.com/pma.transfer/doku.php?id=context\\_menu\\_for\\_slides&rev=1589529290](https://docs.pathomation.com/pma.transfer/doku.php?id=context_menu_for_slides&rev=1589529290)

Last update: **2020/05/16 05:14**

