

Origin server	Origin path	Target server	Target path	File size	Status	Progress	Time

Queued upload files   Queued download files   Aborted transfers   Successful transfers   **Server to server transfers**

The table view is intended to monitor transfer progress and keep track of transfers' outcome.

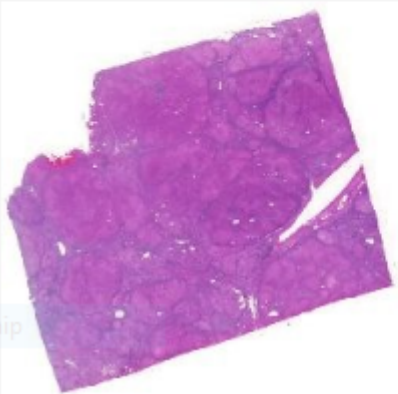
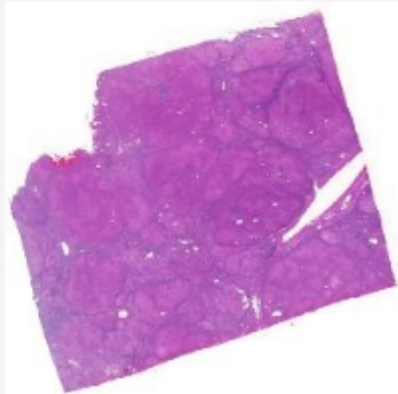
When a transfer is started, the slide is added to the view in a table view fashion. The status and the progress bar for each slide keep the end user informed about the transfer progress.

Origin server	Origin path	Target server	Target path	File size	Status	Progress	Time
https://host.pathomation.com/...	wsiformats/brightfield/Zeiss/jp...	My Pathomation (https://my-fr...	P0061_767CJN/folder1/jpeg_r...	3,426,819,648	Running...	<div style="width: 20%;"></div>	

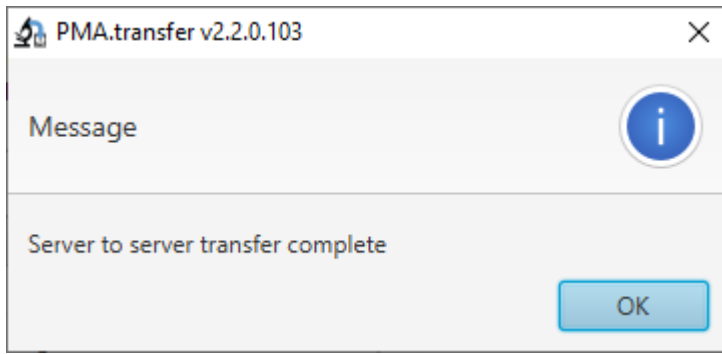
It's worth adding that since this is a server to server transfer, the speed of the operation is **way faster** than the one for an upload or a download, therefore the transfer of some big slides (> 1GB) can take very little time.

Once the transfer is complete, an integrity check takes place behind the scenes, in case the fingerprint doesn't match, the integrity window pops up (a red font is used to mark the incoherence on the fingerprint)

**Thumbnail integrity check for [jpeg\_rgb\_pyramidal.czi]** ✕

<p>Slide at origin:</p>  <table border="1" style="width: 100%; border-collapse: collapse;"> <tr><td># of ZoomLevels</td><td>9</td></tr> <tr><td>Fingerprint</td><td>d7wn8Pyd8kPX4</td></tr> <tr><td>Slide size</td><td>3,426,819,648 b</td></tr> <tr><td>Source folder</td><td>wsiformats\brig</td></tr> </table>	# of ZoomLevels	9	Fingerprint	d7wn8Pyd8kPX4	Slide size	3,426,819,648 b	Source folder	wsiformats\brig	<p>Transferred slide:</p>  <table border="1" style="width: 100%; border-collapse: collapse;"> <tr><td># of ZoomLevels</td><td>9</td></tr> <tr><td>Fingerprint</td><td>hOlq19NDzyQJ</td></tr> <tr><td>Slide size</td><td>3,426,819,648 b</td></tr> <tr><td>Target folder</td><td>P0061_767CJN\</td></tr> <tr><td>Transfer duration</td><td>00:00:00</td></tr> </table>	# of ZoomLevels	9	Fingerprint	hOlq19NDzyQJ	Slide size	3,426,819,648 b	Target folder	P0061_767CJN\	Transfer duration	00:00:00
# of ZoomLevels	9																		
Fingerprint	d7wn8Pyd8kPX4																		
Slide size	3,426,819,648 b																		
Source folder	wsiformats\brig																		
# of ZoomLevels	9																		
Fingerprint	hOlq19NDzyQJ																		
Slide size	3,426,819,648 b																		
Target folder	P0061_767CJN\																		
Transfer duration	00:00:00																		

Once all transfers are complete, a window pops up to keep the end user informed that all transfers are over :



For complete transfers, the end user have the possibility to launch the integrity check window (this can be done via mouse right click on the root of the slide or directly via a double click on that same root).

Origin server	Origin path	Target server	Target path	File size	Status	Progress	Time
https://host.pathomation.com/...	wsiformats/brightfield/Zeiss/jp...	My Pathomation (https://my-fr...	P0061_767CJN/folder1/jpeg_r...	3,426,819,648	Complete	<div style="width: 100%;"></div>	Sat Sep 05 03:28:25 WAT 2020
https://host.pathomation.com/...	wsiformats/brightfield/Zeiss/ra...	My Pathomation (https://my-fr...	P0061_767CJN/folder1/raw_rg...	7,612,928	Complete	<div style="width: 100%;"></div>	Sat Sep 05 03:55:49 WAT 2020
https://host.pathomation.com/...	wsiformats/brightfield/Aperio/...	My Pathomation (https://my-fr...	P0061_767CJN/folder1/aperio...	289,250,433	Complete	<div style="width: 100%;"></div>	Sat Sep 05 03:57:55 WAT 2020

An integrity check windows then pops up to compare both slides on sender/receiver remote sites :

**Thumbnail integrity check for [JP2K-YCBCR-2.svs]**

Slide at origin:

Transferred slide:

# of ZoomLevels	8
Fingerprint	rVNq3mQbzLnv
Slide size	289,250,433 byt
Source folder	wsiformats\brig

# of ZoomLevels	8
Fingerprint	rVNq3mQbzLnv
Slide size	289,250,433 byt
Target folder	P0061_767CJN\
Transfer duration	00:00:00

From:  
<https://docs.pathomation.com/pma.transfer/> - **PMA.transfer**

Permanent link:  
[https://docs.pathomation.com/pma.transfer/doku.php?id=monitoring\\_a\\_transfer&rev=1599275271](https://docs.pathomation.com/pma.transfer/doku.php?id=monitoring_a_transfer&rev=1599275271)

Last update: **2020/09/05 06:07**

