

Release history

Version	Revision	Released
PMA.transfer 2.0.0	50	May 2019
PMA.transfer 2.0.1	55	July 2019
PMA.transfer 2.0.2	60	September 2019
PMA.transfer 2.0.3	72	January 2020
PMA.transfer 2.1.0	85	May 2020
PMA.transfer 2.1.1	87	June 2020
PMA.transfer 2.2.0	103	August 2020
PMA.transfer 2.2.1	111	February 2021
PMA.transfer 2.2.2	114	July 2021
PMA.transfer 2.2.3	9d6e296f92	August 2021
PMA.transfer 2.2.4	d67f7abd23	February 2021

What's new in version 2.0.1

General improvements :

- Log file menu item to open directly the current log file in addition to log files' location
- Added file format to “inspect slide” interface

Bug fixes :

- VSI slides' transfer issue
- Dealing with leading/trailing spaces in host & username text fields and trailing slashes for host as well

What's new in version 2.0.2

General improvements :

- Ask for confirmation before closing the app
- Auto detect white or black text color depending on text area background color
- Add recursive folder download feature

Bug fixes :

- Fix thumbnail URL encoding issue on local site
- Tests against all supported file formats, bug fixing

What's new in version 2.0.3

New features :

- Bypass proxy
- Delete directories on server side (only if empty and not a root one)

- Rename directories on server side (only if empty and not a root one)
- Delete slides on server side
- Rename slides on server side
- Add copy full path option both locally and remotely

General improvements :

- Directly download to selected folder on local site
- Keyboard input interaction
- New icon for root directories
- Better transfer speed metrics
- Better error messages
- Enhanced integrity check
- Adding counters to all bottom bar buttons

Bug fixes :

- Refresh table view for local site following successful downloads

What's new in version 2.1.0

New features :

- Interfacing with the new My Pathomation API
- Adding create/rename/delete folder features for My Pathomation accounts
- Adding rename/delete slide features for My Pathomation accounts
- Adding drag & drop functionality for download
- Release of a self-containing package for Windows (No installation of JRE required)

General improvements :

- Detection of PMA.core version on connection
- Harmonization of upload/download UI/UX
- Cleaning up UI for slides without barcode
- Defining explicit location while adding new folders

Bug fixes :

- Disabling of upload/download buttons on disconnection

What's new in version 2.1.1

General improvements :

- Compliance with PMA.start and PMA.start dotnetcore on Windows
- Adding support of VPN connections

What's new in version 2.2.0

New features :

- Server to server transfers
- Multiple selection of slides for transfer

General improvements :

- Compliance with PMA.start dotnetcore on MacOS & Unix
- Improved error messages (transfer of empty folders, transfers to location that require elevation of privileges..)
- Improved integrity check process

Bug fixes :

- Better handling of failed connections
- Fixes on the site manager

What's new in version 2.2.1

New features :

- User quota enforcement

General improvements :

- Error handling for uploads to Azure directories
- Improved error handling for server to server transfers
- Support for My Pathomation "add slide" API endpoint & "caller ID" parameter for slides uploaded via PMA.transfer
- Support for Keyboard key <Enter>

Bug fixes :

- Fixes on the server to server progress bar

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
What's new in version 2.2.2

New features :

- Improved reporting of successful slide transfers (interactive vs. console)

Bug fixes :

- Slide (file)size is reported correctly again
- Connectivity to My Pathomation is operational again



From:

<https://docs.pathomation.com/pma.transfer/> - **PMA.transfer**

Permanent link:

https://docs.pathomation.com/pma.transfer/doku.php?id=release_history&rev=1677499209

Last update: **2023/02/27 15:00**

