

# Toolbars

## Toolbar



From left to right :

- Site Manager : launches site manager window
- Disconnect : disconnect from current session, if no connection is already established an alert dialog pops up for notification.
- Start upload : Enabled when a slide on local HDD is selected, it initiates an upload to the remote site PMA.transfer is actually connected to (a couple of checks are operated before an upload can be started : duplicate slide name, slide already being uploaded, no connection...)
- Stop upload : Enabled when there are active uploads running, it aborts all current running uploads.
- Start download : Enabled when a slide on the remote site is selected (the remote site on the right that PMA.transfer is actually connected to), it initiates a download to the local site (a couple of checks are operated before a download can be started : duplicate slide name, slide already being downloaded, no connection...)
- Stop download : Enabled when there are active downloads running, it aborts all current running downloads.
- Clear log : Clears logging messages in the text area below connection text fields
- Switch Mode : To switch between a PMA.core server and Pathomation cloud (it operates the necessary disconnections if required)

## Connection bar & logging text area

For PMA.core :

Host:  Username:  Password:  Folder:

status: Trying to connect...  
status: Resolving address of   
status: Connecting to   
status: PMA.core v2.0.0.1686  
status: Connection established...  
status: Loading folder on PMA.core...  
status: Folder successfully loaded...

- Host : PMA.core instance to connect to, it should be in this form "http(s)://xxxxx/pma.core.2/"
- Username & Password : connection credentials
- Folder : Remote directory on server to navigate to if connection is successful

For My Pathomation :

Email:  Password:  Folder:

status: Trying to connect to the cloud...  
status: Resolving address of   
status: Connecting to   
status: PMA.core v2.0.0.1696  
status: Connection established...  
status: Loading user's folder on the cloud...  
status: Folder successfully loaded...

- Email : The email address used to subscribe to My Pathomation (Pathomation cloud).
- Password : Password received on inbox once the subscription & payment are validated.
- Folder : disabled text field, end user can only connect to its own subscription folder.

From:

<https://docs.pathomation.com/pma.transfer/> - **PMA.transfer**

Permanent link:

<https://docs.pathomation.com/pma.transfer/doku.php?id=toolbars>

Last update: **2021/06/18 17:13**

