

What's new in version 2.2.3?

PMA.transfer 2.2.3.9d6e296f92 is an improved version of the previous PMA.transfer with fixed bugs and new features in functionality.

General improvements

- Upload slides and folders to the Amazon s3 cloud.
- Upload slides and folders recursively.
- Displaying information about slides in the cloud directories window.
- Adding a new folder in the cloud directory.

New features in functionality

- Added the possibility to fill folders with slides and subfolders recursively, this means that you can load a folder containing subfolders containing both folders and slides.
- Added the possibility to unload multiple selected slides.
- Added the possibility to delete several selected slides from s3 the cloud directory at once.
- Added the possibility to create a new folder in the cloud S3 directory.
- Added display in the S3 cloud directory of slide data such as: Slidesize, FileType, LastModified.

Bug fixed

- Bug fixed preventing more than one slide from being uploaded.
- Bug fixed preventing more than one slide from being deleted.
- Bug fixed preventing more than one folder from being deleted.
- Bug fixed preventing making a new folder in the cloud directory.

Slide formats

Slide formats with PMA.transfer has been tested:

- * .mrxs
- * .svs
- * .ndpi
- * .vsi
- * .jpg
- * .png
- * .zif
- * .tiff

Last update: 2022/09/08 12:57 what_s_new_in_version_2.2.3 https://docs.pathomation.com/pma.transfer/doku.php?id=what_s_new_in_version_2.2.3&rev=1662631038

From: <https://docs.pathomation.com/pma.transfer/> - **PMA.transfer**

Permanent link: https://docs.pathomation.com/pma.transfer/doku.php?id=what_s_new_in_version_2.2.3&rev=1662631038

Last update: **2022/09/08 12:57**

